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EN Instructions for use

Product description and composition

HydroClean is a hydroactive wound dressing that contains, as a core component, a superabsorbent polyacrylate (SAP) embedded in cellulose fibers and activated (premoistened) with Ringer’s solution. The wound contact layer of the dressing is made up of a knitted fabric, consisting of polypropylene, to which silicone strips have been applied. The wound contact layer and the silicone strips counteract adhesion to the wound bed. On the side facing away from the wound, a water-impermeable polypropylene film, coated with a hydrophobic polypropylene non-woven fabric, prevents the wound dressing pad from drying out too early. This enables moisture to be delivered to the wound bed for up to three days, whilst the polypropylene film also prevents the outer surface of the dressing from being soaked.

Properties and mode of action

HydroClean delivers Ringer’s solution to the wound for up to three days. During this time, interactive and continuous wound irrigation takes place, due to the wound dressing pad also absorbing wound exudate. The SAP used in HydroClean inactivates matrix metalloproteinases (MMP) which may impair wound healing.

Intended use

HydroClean is suitable for wet/moist treatment of wounds. The dressing should be used by healthcare professionals or laypersons trained by healthcare professionals.

Indications

HydroClean is indicated for the wet management of dry, light and moderately exuding wounds, in particular of wounds with impaired healing tendency. The product is especially suitable for the treatment of chronic and poorly healing wounds during the cleansing and granulation stage.

Mode of application and dressing change

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Contraindications

Do not use HydroClean on patients with an intolerance to any of its components.

Side effects

At the start of the treatment with HydroClean, breakdown of irreversibly damaged tissue can lead to an increase in the size of the wound. This can be a sign that wound healing has commenced. Reddening of the wound edges can occur during wet/moist treatment; this is normally a sign that blood circulation has been reactivated as the wound begins to heal. However, the cause must be assessed individually in each case. HydroClean therapy itself does not cause any pain. However, an open wound is often highly sensitive to mechanical stimuli. The question whether an adequate analgesic therapy is indicated must be considered on a case-by-case basis. Whilst wet/moist treatment with Ringer’s solution does not lead to maceration of vital cells, it can cause swelling of the soaked plantar horny layer, which then appears as a whitish coating. In rare cases, unspecific activation of gluteal and leg muscles had been observed in paraplegic patients with sacral wounds. The symptoms were immediately resolved after the removal of the dressing.

Product disposal

To minimize the risk of potential infection hazards, or environmental pollution, disposable components of the medical device should follow disposal procedures according to applicable and local laws, rules, regulations and infection prevention standards.

Special instructions

Re-use of the product may cause skin irritations, a prolonged healing time, pain, and/ or infections. Reprocessing devices, in order to reuse them, may seriously damage their integrity and their performance.

Incident reporting

For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation (EU) 2017/745 on Medical Devices), if during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority. Date of revision of the text: 2022-11-17

AU – PAUL HARTMANN Pty. Ltd. - Macquarie Park, NSW 2113
GB – PAUL HARTMANN Ltd. - Heywood/Greater Manchester OL10 2TT
ZA – HARTMANN South Africa - Northriding, Johannesburg, Gauteng

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FR – Laboratoires PAUL HARTMANN S.A.r.l. - Châtenois – 67607 Sélestat CEDEX
IT – PAUL HARTMANN S.p.A. - 37139 Verona