**1** Schutzkappe | protective cap | capuchon de protection | coperchio di protezione | tampa protectora | tampa protetora | beschermkap | pokrywa ochronna | fodal

**2** Heizplättchen (Anwendungsteil) | heating plate (applied part) | plaque chauffante (partie appliquée) | piastra termica (parte applicata) | placa calefactora (superficie para tratamiento) | placa de aquecimento (área de aplicação) | verwarmingsplaat (toepassingsdeel) | płytka grzewcza (obszar zastosowania) | värmeplatta (användningsyta)

**3** USB-C / Lightning Stecker | plug | connector | connectore | encheuf | plugue | connector | złącze | kontakt

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**IP42**  
Die Umverpackung besteht aus Pappe. Bitte umweltgerecht entsorgen. The outer packaging is made of cardboard. Please dispose of it in an environmentally friendly manner. Le suremballage est en carton. Veuillez l'éliminer dans le respect de l'environnement. L'imballaggio esterno è in cartone. Si prega di smaltirlo nel rispetto dell'ambiente. El embalaje exterior es de cartón. Elimínelo de forma respetuosa con el medio ambiente. A embalagem exterior é feita de cartão. Por favor, elimine-a de uma forma amiga do ambiente. De buitenverpakking is van karton. Gooi het op een milieuvriendelijke manier weg. | Opakowanie zewnętrzne jest wykonane z kartonu. Prosimy o utylizację w sposób przyjazny dla środowiska. | Ytterförpackningen är tillverkad av kartong. Bänligen kassera den på ett miljövänligt sätt. Das Gerät hat eine voraussichtliche Betriebsdauer von 2 Jahren. Nicht über den Produktmüll entsorgen. Entsorgung gemäß Elektro- und Elektronik-Altgeräte Richtlinie der europäischen Gemeinschaft. - WEEE an Sammelstellen für Elektro- und Elektronikschrott. Adressen und Öffnungszeiten der Sammelstellen erfahren Sie bei Ihrer örtlichen Verwaltung. | The device has an expected service life of 2 years. Do not dispose with household waste. Dispose of it in accordance with the Waste Electrical and Electronic Equipment Directive of the European Community - WEEE at collection points for electrical and electronic waste. Addresses and opening hours of the collection points can be obtained from your local administration. | L'unità è a vita prévue de 2 ans. Ne pas jeter avec les ordures ménagères. Il dispositif a une durée prévue de 2 ans. Non gettare con i rifiuti domestici | El dispositivo tiene una vida útil prevista de 2 años. No disponer del dispositivo con los residuos de basura domésticos. | O dispositivo tem uma vida útil prevista de 2 anos. Não se desfacam do lixo doméstico. | De eenheid heeft een verwachte levensduur van 2 jaar. Niet met het huisvuil weggoien. | Przewidywany okres eksploatacji urządzenia wynosi 2 lata. Nie wyrzucać razem z odpadami domowymi. | Anordningen har en förväntad livslängd på 2 år. Kastas ej i brännbart avfall.

**1. Intended Use & Operating Principle**  
**heat it** is a medical device to treat itching and pain caused by insect bites and stings, for example caused by:

- mosquitos
- horseflies
- wasps
- bees

**heat it** works by locally heating an affected skin region to temperatures of about 50°C for a few seconds ("hyperthermia"). The application can relieve itching and pain quickly and for a long time. Treatment is possible both immediately after the sting and if symptoms occur later.

**2. Installation**  
**heat it** only works in conjunction with a suitable smartphone with CE marking.

1. Scan the QR code or search for the **heat it** app in your app store.
2. Install the **heat it** app (Version 2.5.0 or newer)
3. Plug **heat it** into the charging port of your smartphone.
4. The app signals that **heat it** can be used.

**heat it** is available in different variants, please note:

- An iPhone 6s to 14 is required to use **heat it Lightning**.
- To use **heat it USB-C**, an Android smartphone (Android 5.0 or newer) or iPhone (15 or newer) is required.
- The smartphone must support the OTG standard. For some smartphones, OTG must be activated via the settings (Oppo, OnePlus, Vivo).
- OTG-capable smartphones with a micro USB port can use the **heat it** with an OTG-capable adapter with CE marking (not included).

**3. Treatment selection**

**Treatment duration:**  
Choose short, medium or long (4 / 7 / 9 seconds)

**Treatment temperature:**  
Choose the **Child** setting (48.5°C) for children and adults with sensitive skin, and the **Adult** setting (51.5°C) for non-sensitive adults. The treatment temperature can be reduced by 1.5°C for **sensitive** skin areas.

In order to get to know the treatment principle, the settings **short**, **child** and **sensitive** must be selected for the first application. For subsequent treatments, the settings should be selected according to personal sensitivity.

**4. Application**  
The app guides you through the simple application of **heat it**:

1. Plug **heat it** into the smartphone. The **heat it** app opens and the LED on the **heat it** pulses green.
2. Select the appropriate settings in the menu, please see **section 3**.
3. Press the start button or a volume button. **heat it** will now preheat for a few seconds. Do not press the **heat it** onto the skin during this time. **Do not press the heat it on your skin during this time**.
4. The completion of the preheating process is indicated on the screen. Holding the smartphone, press the **heat it** heating plate onto the bite. To stop the treatment prematurely, lift **heat it** off the skin.
5. The end of the treatment is indicated on the screen. Lift the heating plate off the bite.
6. Although this will usually not be necessary, the treatment can be repeated after a pause of **at least 2 minutes**.
7. To switch off **heat it**, disconnect it from the smartphone.

**5. Other Information**

- **Remove** bee stinger before treatment with **heat it**.
- **heat it** may be used with the assistance of an adult on children aged 3 and older. It can be used by children age 12 and above.
- In case of any uncertainty and for the first application, the symbols **Short**, **Child** and **Sensitive** must be selected. The same applies for the treatment of body parts that have not yet been tested.
- Special maintenance of **heat it** is not necessary, but the heating plate should be cleaned regularly with a disinfecting cloth. Disconnect **heat it** from the smartphone before cleaning.
- Do not clean the **heat it** under running water or immerse it in water.
- A red pulsating LED indicates that the **heat it** cannot connect correctly to the app. Please ensure that a current version of the app is installed and that the **charging socket is free of dirt**.
- **heat it** is a medical electrical equipment which becomes a medical electrical system in combination with the smartphone.
- Please make sure that the **heat it** is firmly seated in the protective cap.
- Only connect **heat it** to CE marked smartphones and OTG adapters if applicable.
- Storage instructions: When not in use for extended periods of time, store **heat it** away from direct sunlight and in a dry place. Disconnect **heat it** from the smartphone and firmly seat it in the protective cap immediately after use and store it separately.

**6. Contraindications**

- Do not use **heat it** on damaged skin (e.g. sunburns) and on wounds.
- Do not use **heat it** on damp skin, or on skin that has previously been treated with ointments, creams or gels.
- **heat it** is designed exclusively for use on the outer skin. Do not use **heat it** in body orifices, on mucous membranes, gums, nail beds, eyes or external auditory canals.
- **heat it** must not be used on particularly sensitive skin regions with poor circulation (poor circulation of the limbs due to diabetes, varicosis, atherosclerosis, scleroderma, Raynaud's symptom, etc.).
- People with permanently or temporarily reduced thermal sensitivity of the skin (e.g., polyneuropathy) must not use **heat it**.

**7. Side-effects**

- While most people find the heat from the heating plate to be tolerable, some may find it unpleasant.
- Excessive use may result in skin injury or persistent discoloration of the skin ("caloric hyperpigmentation").
- Sensitive people may experience a brief reddening of the skin after use. In rare cases, sensitive users may experience skin irritation.

**8. Warnings**

- The treatment with **heat it** does not replace a medical consultation and treatment. **heat it** is not suitable for treating allergic reactions (anaphylactic reactions). In case of any kind of pain or illness and if any doubt a doctor must be consulted.
- **heat it** cannot prevent disease transmission or cure symptoms or diseases that are not listed in the intended use. If there are indications of a transmitted disease, a doctor must always be consulted.
- If you have any doubts about the use of **heat it**, be sure to consult a doctor.
- The same skin area must not be treated more than five times per hour.
- A break of at least 2 minutes is compulsory between repeated treatment on the same location.
- Do not press **heat it** on the skin during the preheating process and never for longer than 10 seconds.
- Keep **heat it** out of reach of children under the age of 12 due to the danger of swallowing small parts!
- In case of unusual behaviour of **heat it**, do not use **heat it** again and contact the manufacturer. Please also report any serious incidents that occur in connection with the **heat it** to the competent authority in your country.
- Do not open or modify **heat it** and do not use if damaged.
- Do not charge the smartphone under any circumstances while using **heat it**. Otherwise, contact with dangerous voltage cannot be ruled out.

Apple, iPhone and Lightning are trademarks of Apple Inc. registered in the U.S. and other countries. Use of the Made for Apple badge means that an accessory has been designed to connect specifically to the Apple product(s) identified in the badge and has been certified by the developer to meet Apple performance standards. Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards. Please note that the use of this accessory with an Apple product may affect wireless performance.

Android and Google Play are trademarks of Google LLC. Design and specifications are subject to change without notice.

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Android e Google Play sono marchi di Google LLC.